

USER GUIDE

EVAC+CHAIR IBEX



IMPORTANT INFORMATION

1. Your Evac+Chair should only be used by trained operators.
2. Operator training on your Evac+Chair Ibex should be conducted in line with Evac+Chair guidelines, as stated in this user guide.
3. Our Operator and Key Trainer courses should be undertaken every 3 years, to ensure competency using the Evac+Chair Ibex.
4. Ensure training records are maintained and up to date (see training record on page 4).
5. Quarterly practice with your Evac+Chair Ibex is recommended to ensure staff remain trained and competent.
6. Quarterly inspection of your Evac+Chair Ibex is recommended, to ensure the integrity of the chair is maintained. See page 20 for information.
7. As a Class 1 Medical Device, Evac+Chair offer service & maintenance of your Evac+Chair that should be undertaken by an approved service provider annually to ensure safe operation. This includes a physical, as well as a visual check and replacement of worn parts.
8. Only genuine Evac+Chair parts should be used when maintaining your Evac+Chair.
9. To ensure the optimal performance of your Evac+Chair, tracks should be rotated every 3 months.

Note: Whilst the products are designed to work in most stair applications, Evac+Chair International accepts no liability for damage to stairs caused when the chair is in use

WARRANTIES & LIABILITIES

The Evac+Chair is guaranteed for five years against defective material and workmanship. This does not include parts subject to wear, such as tracks, or damage caused by abuse or sabotage, and/or any other damage which has occurred due to a breaching of the guidelines contented in the User Guide or Training Manual. This includes getting your Evac+Chair serviced by a third party company. Tracks are advised to be changed every 2 years on a basis of average use and practice.

Neither the manufacturer nor their agents shall make recompenses for repairs carried out by third parties or be liable for any damage, either incidental or consequential to property or persons. If you, the customer, are dissatisfied with the product or service provided by Evac+Chair International, please contact our customer service team on +44 (0)121 706 6744 or enquiries@evacchair.co.uk.

CONTENTS

4-5 Training & Maintenance Records

6-8 General Information

9-10 Deployment

11-12 Transfer

13-16 Ascent/Descent

17-18 Training & Maintenance

19 Wall Bracket Installation

20 Visual Inspection

21 Accessories

22 International User Guide Online



International User Guide Online
evacchair.co.uk/user-guides



TRAINING RECORD

Trainee Name	KT / OT*	Training Date	Expiry Date	Trained by

* KT = Key Trainer * OT = Operator Trainer

MAINTENANCE RECORD

Date	Service or Inspection	Tracks Changed	By	Company

NOTE: Tracks should be rotated every 3 months

THE EVAC+CHAIR IBEX



Intended Use: The Ibex is a robust two-person operation evacuation chair. Suitable for spiral and narrow staircases. Please read this user guide in full. Should you have questions, please contact your Evac+Chair Account Manager or authorised Evac+Chair distributor for more information.


SAFE WORKING LOAD



FOR ASCENT AND DESCENT



SUITABLE FOR

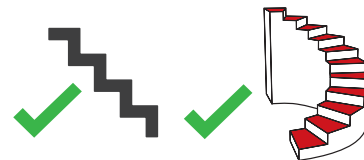

MAINTENANCE (STRONGLY ADVISED)



TRAINING (STRONGLY ADVISED)



MANUFACTURERS WARRANTY




ADVISORY




Two-person operation

STAIR ANGLE

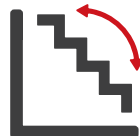

28° — 40°

MINIMUM ALLOWABLE STAIR WIDTH

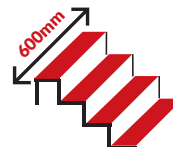

WHAT IS INCLUDED?

CHAIR



USER GUIDE



DUST COVER






WALL BRACKET






PHOTOLUMINESCENT SIGN




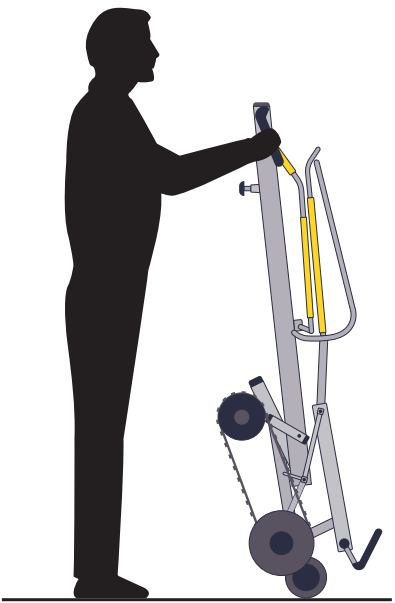

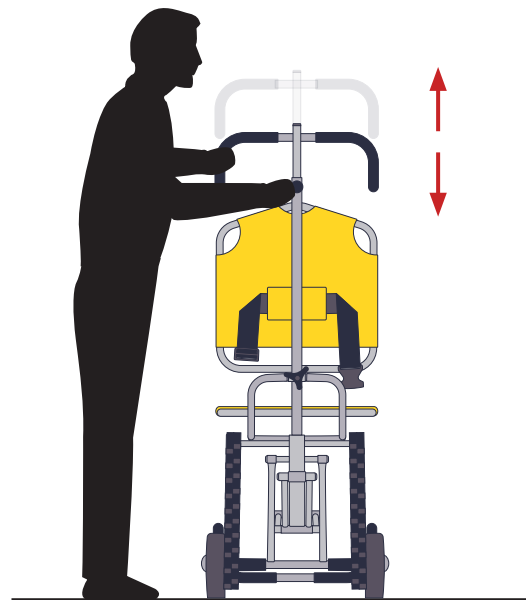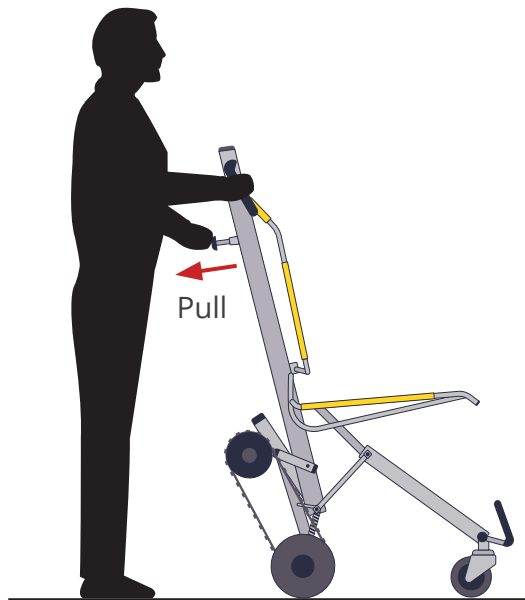


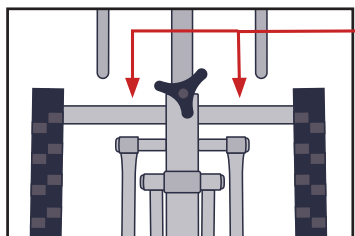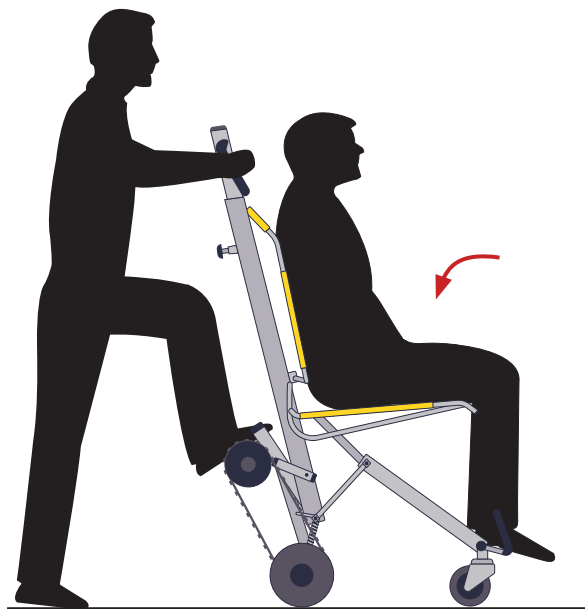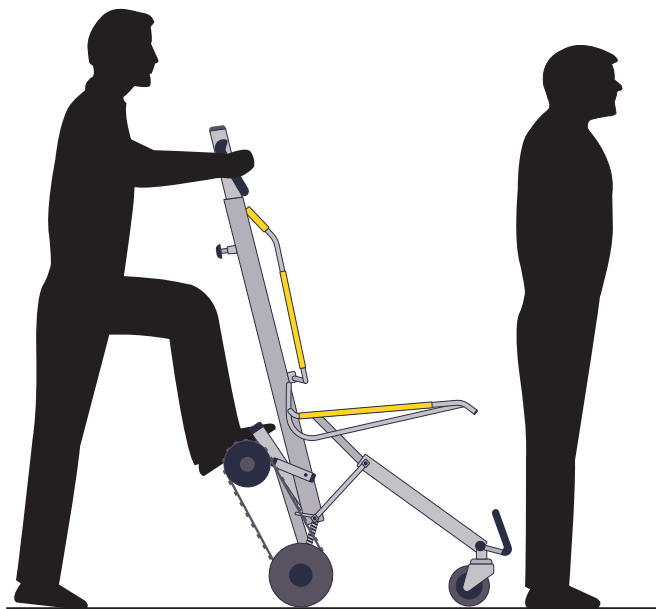



Operator
foot position

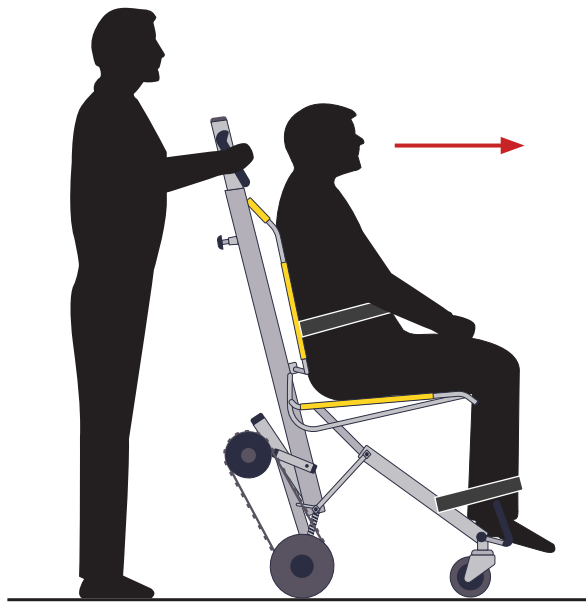


X2

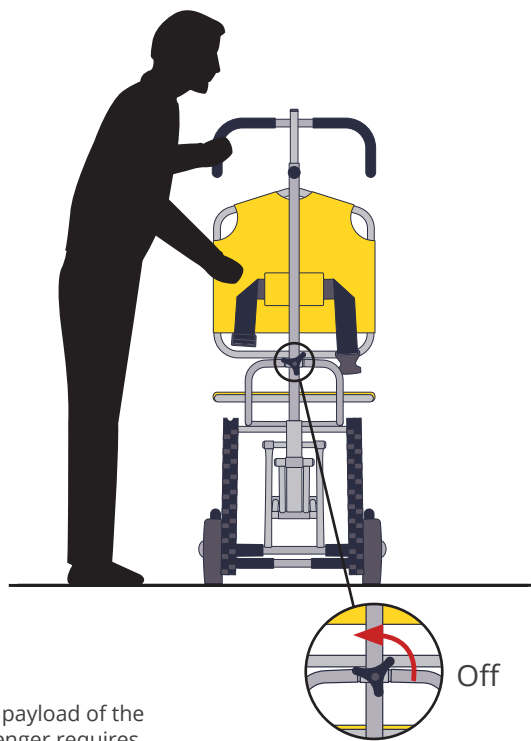

Adjust the brake system to suit the payload of the passenger. A heavier payload passenger requires greater brake application.

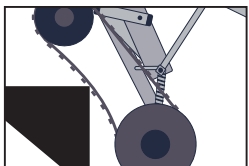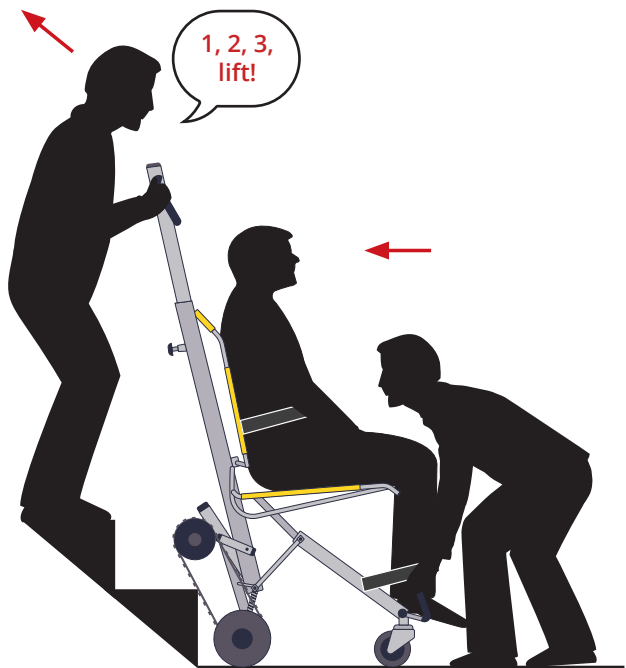

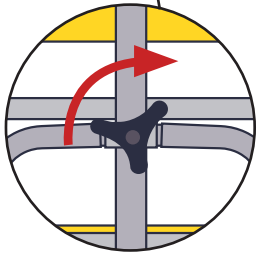

On

Adjust the brake system to suit the payload of the passenger.
A heavier payload passenger requires greater brake application.




Tilt chair and passenger back approx. 10°.



TRAINING

DO YOU HAVE TRAINED OPERATORS?

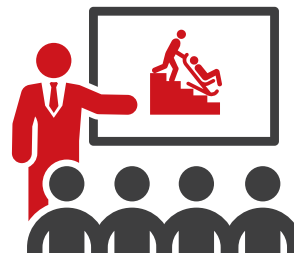

REASONS TO BE TRAINED:

- 1** To adhere with legislation such as the Regulatory Reform Order 2005 (UK Only).
- 2** Renew your three year certification for chair use.
- 3** Training multiple employees means more people are able to operate and assist.
- 4** Be able to confidently operate the chair in your premises.
- 5** Ensure you comply with your evacuation policy.

Operator Training Course

A 2.5-hour training course designed to train delegates how to safely and correctly use an Evac+Chair. The course enables delegates to have hands-on experience in operating and deploying the Evac+Chair and provides guidance on the safe evacuation, of the mobility impaired.

Key Trainer Course

A 4-hour training course designed to train delegates how to safely and correctly operate an Evac+Chair, as well as providing the practical and theoretical skills to train and certify others. This course is designed for organisations with large numbers of people who require the flexibility to train in house.

Our training courses are suitable for 2-4 delegates. On successful completion, each delegate will be certified for three years.


To discuss your training requirements, please call
+44 (0) 121 706 6744

evacchair.co.uk/training




SERVICE & MAINTENANCE

HAVE YOU ARRANGED A SERVICE AND MAINTENANCE CONTRACT?

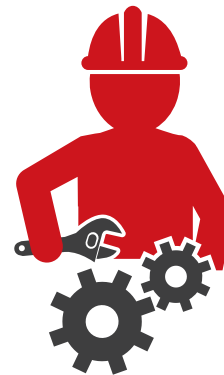

As a Class 1 Medical Device, your Evac+Chair should be regularly serviced and maintained to ensure its safe operation. This requirement is included within the PUWER Regulation (Provision and Use of Works Equipment Regulations 1998 - UK only).

Our maintenance contracts provide you with complete peace of mind that you are covered and should you have an emergency evacuation situation, your chair will be fully operational. Our contracts include labour and genuine replacement parts, ensuring your Evac+Chair is kept fully operational and meets legal requirements.

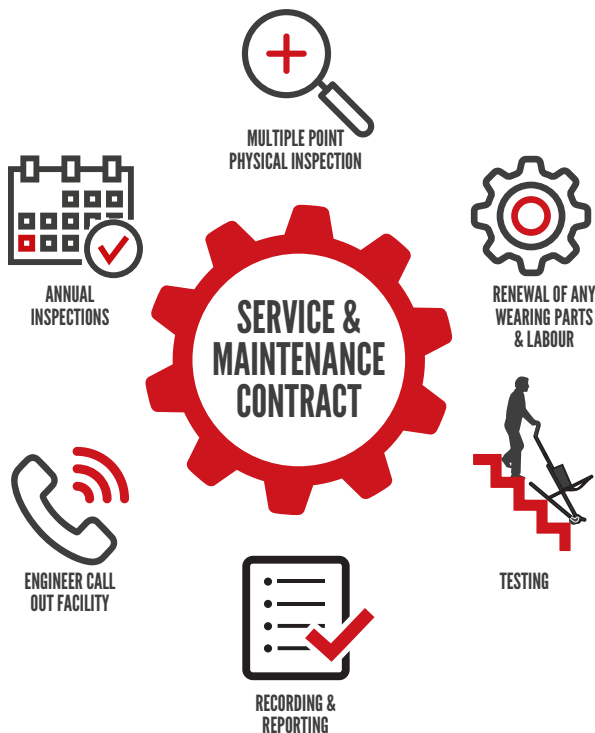


To discuss your maintenance requirements, please call
+44 (0) 121 706 6744

evacchair.co.uk/maintenance



WALL BRACKET- FOR WALL MOUNTING

Back

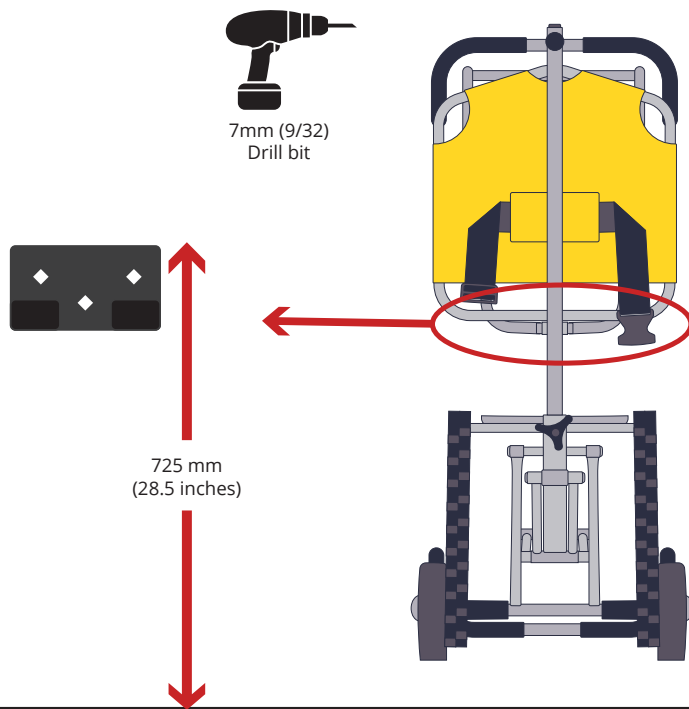

INSTALLATION



Evac+Chair, or an authorised distributor, can install your evacuation chair as well as provide expert advice and guidance on the most effective locations within your building.



For storage, ensure the Ibex is in its folded position, lift and place the seat frame onto wall hooks.



evacchair.co.uk/maintenance

VISUAL INSPECTION

WE RECOMMEND YOU CARRY OUT A QUARTERLY INSPECTION TO ENSURE THE INTEGRITY OF THE CHAIR IS MAINTAINED.

DEPLOYMENT/STORAGE



- + Ensure chair deployment is smooth
- + No interference when chair is deployed or stored
- + Ensure chair closes correctly

CHECK WHEELS



- + No cracking of wheel or hub
- + No splitting of wheels
- + Ensure wheels are free to rotate without impediment

CHECK SEATING



- + No obvious threading around seams
- + No ripping of hammock material
- + Check belt buckle operates correctly

CHECK TRACKS



- + Rotate tracks every 3 months
- + No deep cracks or rips
- + No bulging



Please contact Evac+Chair, or an authorised distributor **immediately** if any of these defects appear on your chair.

ACCESSORIES

PHOTOLUMINESCENT SIGN



PATIENT CARRY SEAT



ANTI-THEFT DEVICE



DUST COVER



For more information on these accessories visit:

evacchair.co.uk/accessories



INTERNATIONAL USER GUIDE ONLINE



To view our user guide online please visit:



Als u onze gebruikershandleiding online wilt bekijken, gaat u naar:



Um unser Benutzerhandbuch online anzuzeigen, besuchen Sie bitte:



Pour consulter notre guide de l'utilisateur en ligne, veuillez visiter:



Per visualizzare la nostra guida utente online, visitare:



Para ver nuestra guía de usuario en línea, visite:



Para ver o nosso guia do usuário on-line, visite:




Aby wyświetlić naszą instrukcję obsługi online, odwiedź:




要在線查看我們的用戶指南，請訪問：



evacchair.co.uk/user-guides

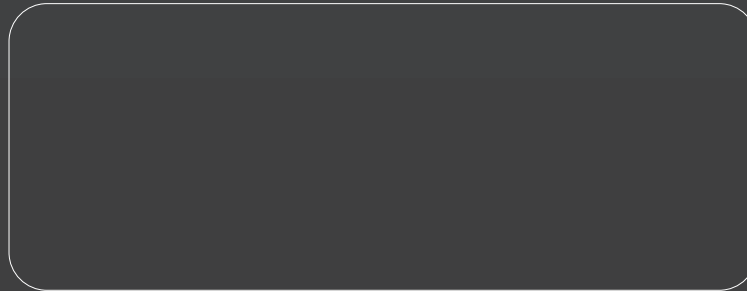

 Evac+Chair International Ltd, Unit 4, Central Boulevard, Blythe Valley Park, Solihull, West Midlands, B90 8AW UK
+44 (0)121 706 6744 | evacchair.co.uk | enquiries@evacchair.co.uk



1-700H-UG-UK-01(V4)

